

CMC740012-K CMC LITTER GUARD RAILING CLAMP KIT

For Ferno 71 Series Stretchers

Fitting Instructions

About the CMC Litter Guard Railing Clamp Kit

The CMC Litter Guard Railing Clamp enables the CMC Litter Shield to be attached to basket stretchers with a rail.

This Clamp Kit enables a drilled and tapped version of the standard Railing Clamp to be attached to the Ferno 71 Basket Stretchers **without drilling holes through the basket shell**.

Kit Inclusions

	DESCRIPTION	QTY
1	CMC Litter Guard Rail Clamp (drilled & tapped)	1
2	Combination Spanner — 8mm	2
3	Flange Nut — 8mm	2
4	Lock Nut — 8mm	2
5	Bolt — 40mm	2



Fitting the Clamp to the Basket Stretcher

- 1. Position the CMC Litter Guard Railing Clamp (refered hereafter as the Clamp) over the 71 Basket Stretcher top edge (Figure 1).
- 2. Thread a 40mm bolt through the bolt hole in the Clamp. Before the bolt end reaches the stretcher's side/skin, thread the lock nut followed by the flange nut onto the bolt.
- Continue threading the bolt through the Clamp until the bolt and flange nut is flush against the basket stretcher's skin (Figure 2).
- 4. Tighten the lock nut against the inside of the Clamp bracket (Figure 2). Verify bolt and nuts are tight and secure.
- 5. Repeat Steps 2 4 to install the second 40mm bolt.
- Proceed with fitting the CMC Litter Shield per manufacturer's instructions and local / organisational protocols.



