Features & Functions Restraints

To Set Up Restraints

- Layout harness on stretcher (Fig. 1)
- Position quick release clip on harness mount point and pull up till it clips into place. (Fig. 2-3). Position and clip black safety plug over harness mount point (Fig. 4-5)
- 3. To fit leg restraints, position on main frame and feed buckle through loop (Fig. 5-7)



Fig 1 Layout restraints



Fig 2 Position Quick Release Clip



Fig 3 Pull up to click into place



Fig 4 Position safety plug over harness mount point



Fig 5 Push and clip into place



Fig 6 Feed Buckle through loop



Fig 7 Restraint connected

Features & Functions (continued)

Restraints (Continued)

Always secure a patient on the stretcher with chest and leg restraints.

To fit restraint to patient

- 1. Unbuckle each restraint and lay out ready for use (Fig. 6).
- 2. Transfer the patient to the stretcher.
- Position the chest straps (Fig. 7) and buckle the waist strap through the slots in the buckles on the end of the chest straps (Fig 8).
- Buckle the chest strap and tighten restraints to safely secure the patient without causing discomfort or impaired circulation.
- 5. Buckle the leg restraint across the patient's legs (Fig. 9).



Fig 7 Chest Straps in Place



Fig 9 Leg Restraint in Place

MARNING

Patients ejected from the stretcher can be injured. Secure restraints around patients placed on the stretcher



Fig 6 Restraints ready for use



Fig 8 Buckle through Slots



Fig 10 Patient Restrained